Close to you.

MANUFACTURER SINCE 1921

reddot design award
winner 2010

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Dear Owner of the Prosthesis Lifter acc. to Dr. Wietzorke,

You and your dentist have rightly chosen a precisely fitting, premium dental prosthesis. Having a new prosthesis with multiple telescopes, it can be especially difficult for you as a patient, to easily remove such a precisely fitting dental prosthesis for cleaning. This is compounded by the fact that a telescope prosthesis can only be removed easily, by uniformly loosening all telescope crowns to avoid the prosthesis getting jammed.

For this purpose, **HELMUT ZEPF** Medizintechnik GmbH has developed the Prosthesis Lifter in co-operation with Dr. Wietzorke. For removing your dental prosthesis correctly by means of the Prosthesis Lifter, please adhere to the following instructions carefully.

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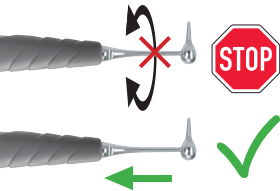
Positioning of the Prosthesis Lifter

Insert the prosthesis lifter with its pointed end between two telescope crowns or between the telescope crown and the prosthesis. Hereby, the handle points in the traction direction, i.e.: in dental prosthesis of the lower jaw: upwards, in dental prosthesis of the upper jaw: downwards.

Attention! The instrument may only be strained in axial direction of tensile force.

Please note, a lateral luxating/rotating movement has to be strictly avoided as this would cause a distortion of the very delicate working tip. We do not assume liability for damage caused by improper handling!

1. Insertion of the Prosthesis Lifter

Push the pointed end in between the crown. The rounded end protects the lip from injuries.

2. Initial Lifting Movement

Slightly pull the handle now, thus loosening the connection between the outer and the inner part of the telescope crown on one side of the jaw.

3. Change Sides

Change sides and insert the prosthesis lifter on the other side of the jaw, between two telescope crowns or between the telescope crown and the prosthesis, and slightly pull the handle of the Prosthesis Lifter. The telescopic joints will now also loosen here.

4. Change Again and Lift

Change sides again and lift again. By repeating this procedure several times, your telescope prosthesis is being released easily and uniformly without getting jammed.

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NOTE: You should first use your Prosthesis Lifter in the dental practice, as per the instructions from your dentist. If you should have difficulties in using it later, please contact your dental team.

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Hersteller | Manufacturer: HELMUT ZEPF Medizintechnik GmbH