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85.903.03

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85.903.04

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# ZEPF Drop-Control® Blade Holder DISMANTLING, CLEANING AND MAINTENANCE

REF 46.007.18

**Dear Customer!**

We thank you very much for having chosen this high quality instrument from Helmut Zepf Medizintechnik GmbH. To get the best from your instrument, please follow the instructions below:

**DISMANTLING**

The ZEPF Drop-Control® Blade Holders have to be dismantled easily **before** cleaning, as shown.

**Caution**

The blade must be removed before disassembly! Don't remove the spring! The instruments may only be used by persons who have been specially trained or instructed for this purpose.

**CLEANING**

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664, in the currently valid version. See document "0703091007\_WIEDERAUFBEREITUNG\_HZ\_BA", downloadable from our website [www.zepf-dental.com](http://www.zepf-dental.com).

**Pre-treatment:** Use cleaning brush with nylon bristles when incrustations are heavy.

**Cleaning:** Manual or mechanical – ATTENTION: Cleaning in a thermal disinfectant: standing vertically, thread downwards.

**Rinsing:** Please use deionized water to avoid water spots.

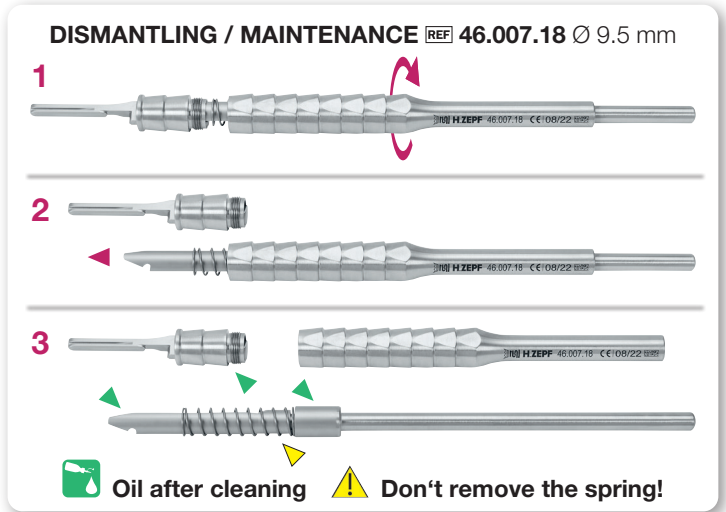
**Drying:** Use compressed air or air-dry (manual preparation).

**MAINTENANCE**

Oiling with Medical White Oil, as shown.

**ASSEMBLY**

Assembly is made in reverse order to the dismantling.

**RECOMMENDED CARE PRODUCTS**

**Medical White Oil:**

24.950.12

Medical White Oil in an Oil Pen, 12 ml

24.950.10

Medical White Oil in a dispensing bottle, 50 ml

**Cleaning Brushes:**

85.903.01 with nylon bristles, sterilizable, PU: 3 pieces

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