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
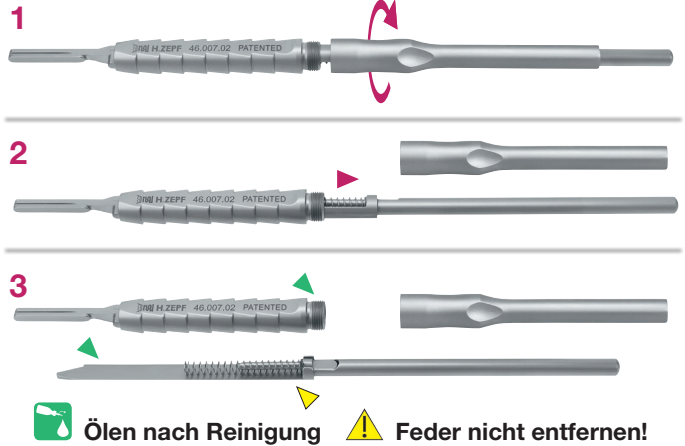
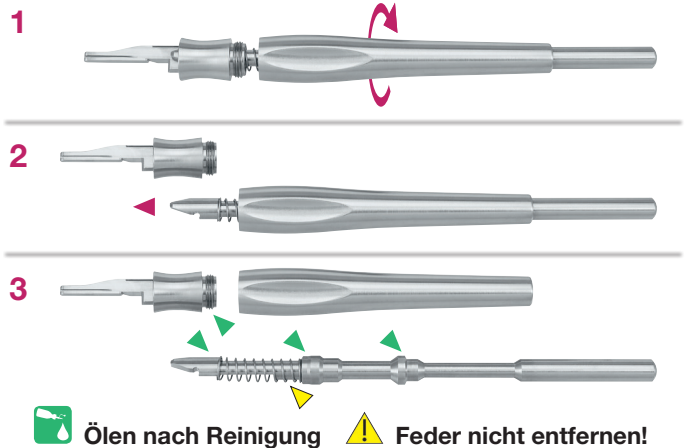
DEMONTAGE / WARTUNG REF 46.007.08 Ø 12 mm

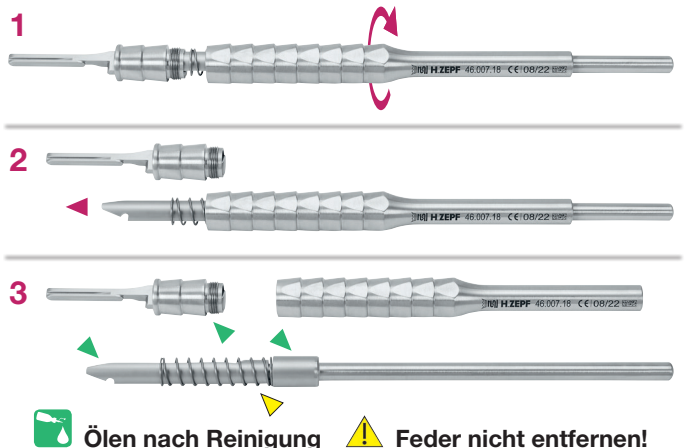DEMONTAGE / WARTUNG REF 46.007.18 Ø 9,5 mm

ZEPF Drop-Control® Klingenhalter DISMANTLING, CLEANING AND MAINTENANCE

REF 46.007.02, REF 46.007.08 and REF 46.007.18

Dear Customer!

We thank you very much for having chosen this high quality instrument from Helmut Zepf Medizintechnik GmbH. To get the best from your instrument, please follow the instructions below:

DISMANTLING

The ZEPF Drop-Control® Blade Holders have to be dismantled easily **before** cleaning, as shown.

Caution

The blade must be removed before disassembly! Don't remove the spring! The instruments may only be used by persons who have been specially trained or instructed for this purpose.

CLEANING

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664, in the currently valid version. See document "0703091007_WIEDERAUFBEREITUNG_HZ_BA", downloadable from our website www.zepf-dental.com.

Pre-treatment: Use cleaning brush with nylon bristles when incrustations are heavy.

Cleaning: Manual or mechanical –
ATTENTION: Cleaning in a thermal disinfectant: standing vertically, thread downwards.

Rinsing: Please use deionized water to avoid water spots.

Drying: Use compressed air or air-dry (manual preparation).

MAINTENANCE

Oiling with Medical White Oil, as shown.

ASSEMBLY

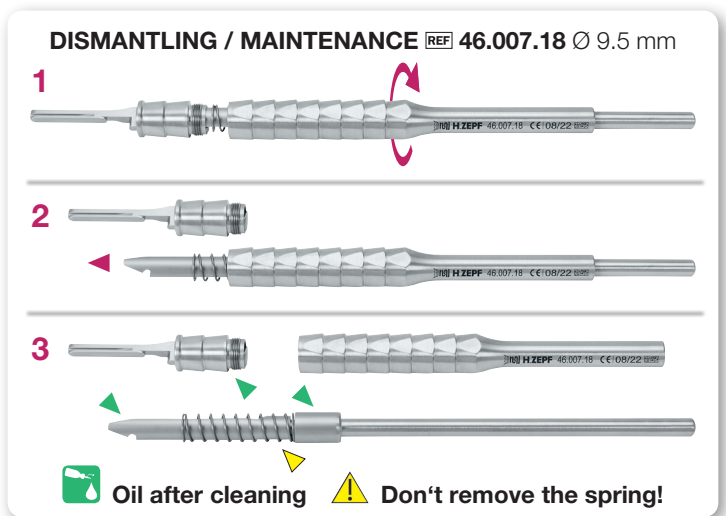
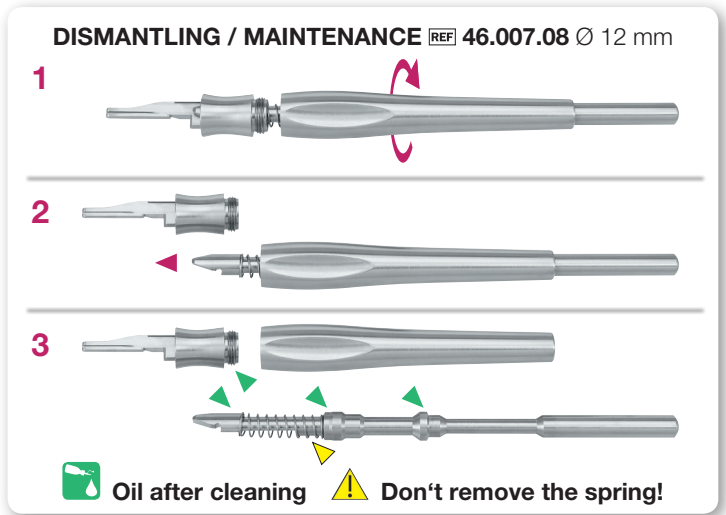
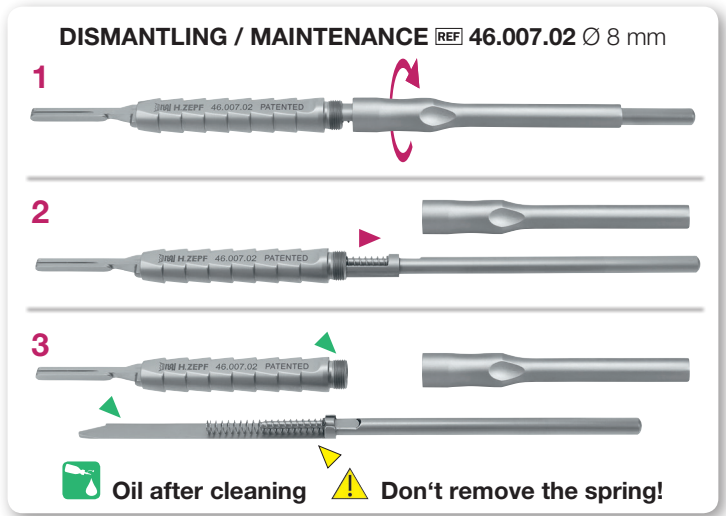
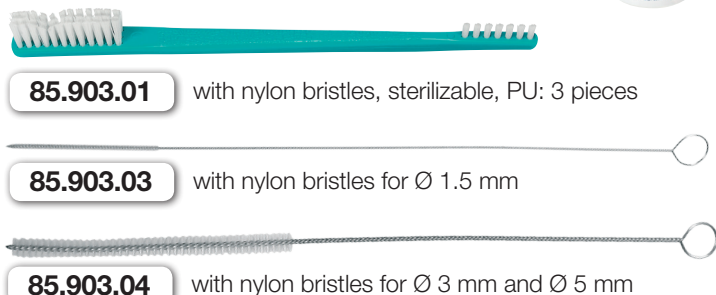
Assembly is made in reverse order to the dismantling.

RECOMMENDED CARE PRODUCTS

Medical White Oil in a 50 ml dispensing bottle REF 24.950.10 and in an Oil Pen REF 24.950.12, 12 ml

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