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### ACHTUNG

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## BASIC CLEANING

### ATTENTION

**Before initial use and sterilization of the instruments, a basic cleaning has to be effected as follows:**

- Disassembly of dismountable instruments.
- Entire, bubble-free insertion in an appropriate detergent. We recommend a commercial dishwashing liquid, taking into consideration the manufacturer's information.
- \*Thorough mechanical cleaning of the entire instrument surface using a smooth plastic brush with nylon bristles (REF 85.903.01 packing unit 3 pieces).
- Rinse the instruments finally in desalinated (demineralized) water in order to avoid spots, residues and corrosion on the instruments.
- Thorough drying of the instruments before sterilization.
- Before initial cleaning, please read the attached instruction for the preparation of resterilizable instruments acc. to DIN EN ISO 17664.

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 \*PROBLEM: **Metal brushes** destroy the passive layer of the instrument surface!

SOLUTION: **Brushes with nylon bristles.**

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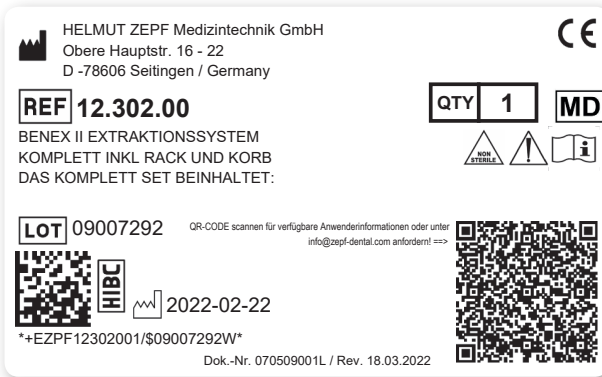
**Dear Customer.**

The new EU Regulation on Medical Devices (Medical Device Regulation) leads to many new requirements. An important new requirement is the information on the packaging label. For this reason, we have updated our label and would like to introduce it to you briefly.

- CE-Zeichen / CE marking
- Hersteller / Manufacturer
- QTY Menge / Quantity
- MD Medizinprodukt / Medical device
- Symbole / Symbols
- REF Artikel-Nr. / Article no. + Artikelbezeichnung / Article description
- LOT LOT Nr. / LOT no.
- HIBC HIBC Barcode + HR Code
- Herstelldatum / Date of manufacture
- \*QR-Code

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**ZEPF-Care** Washbasket – the Space Wonder  
Example: equipped washbasket 1/1

85.195.00

**ZEPF Dental Katalog**

[https://www.zepf-dental.com/wp-content/uploads/PDF/katalog/HZ\\_Dental\\_Katalog-D.pdf](https://www.zepf-dental.com/wp-content/uploads/PDF/katalog/HZ_Dental_Katalog-D.pdf)

**ZEPF Dental Katalog**

[https://www.zepf-dental.com/wp-content/uploads/PDF/katalog/HZ\\_Dental\\_Catalog-E.pdf](https://www.zepf-dental.com/wp-content/uploads/PDF/katalog/HZ_Dental_Catalog-E.pdf)