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Metal Syringe, self-filling, with Luer Lock Cannula

ZEPF REF 19.750.00

Dear Customer!

We thank you very much for having chosen this high quality product from Helmut Zepf Medizintechnik GmbH. To get the best from your instrument, please follow the instructions below:

IMPORTANT!

- The application syringe has to be cleaned and sterilized before use.
- Please make sure that this kind of syringe is only used by authorized qualified personnel.
- Cleaning and Sterilization: After use with a patient and prior to further use, the syringe has to be disassembled, cleaned and sterilized (in the autoclave at max. 134° C / 273° F).

COMPONENTS OF METAL SYRINGE

- Corpus
- Luer Lock Cannula, as accessory **REF 19.762.00**
- O-Ring, as spare part **REF 19.750.01**

HANDLING

- Connect the corresponding front part needed for the applications by means of the Luer-Lock-fitting onto the syringe and tighten it. The syringe is now ready for use.
- To avoid stiffness, the O-ring has to be lubricated before the first use and after each disinfection and before sterilization by means of medical white oil / instrument oil.

REPROCESSING – SPECIAL FEATURES

- Remove obvious dirt particles from syringes immediately after utilization. It is not allowed to use fused agents or hot water (> 40° C) for cleaning as these agents lock residues and impede an effective purification process.
Attention: Maximum exposure time: 15 minutes!
- **Manual pre-cleaning / cleaning:** In disassembled condition. Drill holes and thread turns must be flushed for at least 10 sec. at a pressure of 3.8 bars by means of a water pistol.
- **Cleaning according to DIN ISO 17664:2004**
- **Drying:** Hollow spaces of the syringes must be dried by means of sterile compressed air.

RECOMMENDED CARE PRODUCTS

-  **Medical White Oil in a 50 ml dispensing bottle and in an Oil Pen, 12 ml**

24.950.10


24.950.12

19.750.01

O-Ring, spare part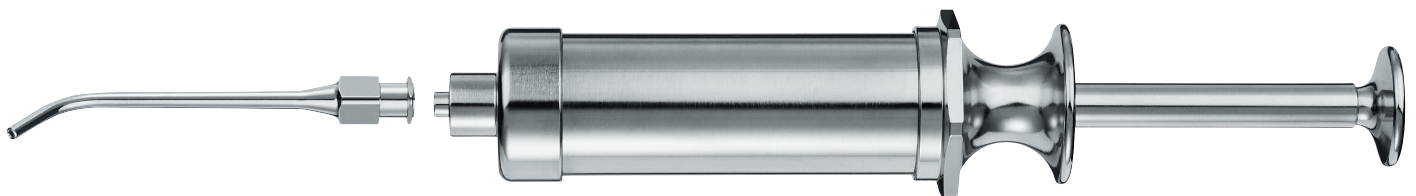

19.762.00

Cannula with Luer Lock Connector, accessory

19.750.00

Metal Syringe, self-filling, with 1 cannula (Luer Lock)