

# Sicherheits-Klingentferner | Security Blade Remover

**ZEPF** REF 46.005.65




www.zepf-dental.com/  
anwenderinformationen  
www.zepf-dental.com/  
en/user-information



## Sehr geehrter Kunde!

Sie haben sich für ein hochwertiges Produkt aus unserem Hause entschieden – vielen Dank.

Damit Sie auch auf Dauer Freude haben, erlauben wir uns, Ihnen nachstehend die Anwendung dieses Instruments zu erklären:

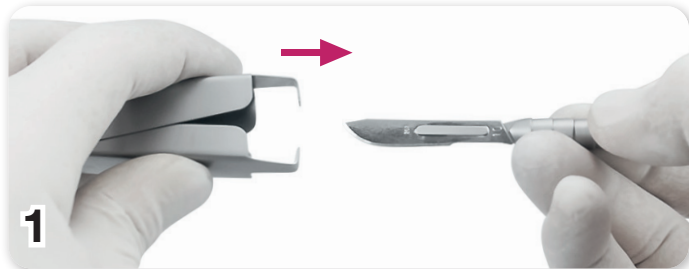


- Skalpellgriff in Klingentferner einführen. Auf richtige Stellung achten. Klingenblock und **ZEPF** Logo auf dem Klingentferner müssen nach oben zeigen.

- Introduce scalpel handle into blade remover. Make sure that scalpel handle and blade remover are in correct position. **ZEPF** logo on blade remover and blade holder on scalpel handle showing upwards.


## Dear Customer!

We thank you very much for having chosen this high quality product from Helmut Zepf.

To get the best from your instrument, please follow the instructions below:

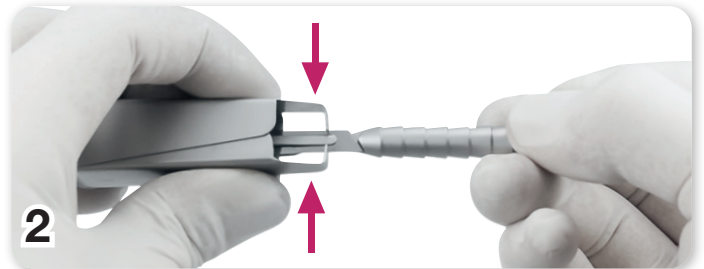


- Schenkel des Klingentferners zusammendrücken.
- Close jaws of blade remover by pressing from both sides.

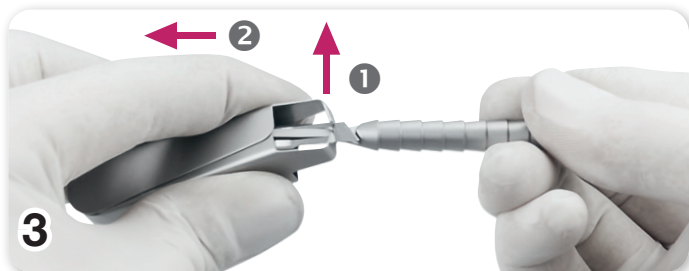

- Skalpellklingenende über die Arretierung des Skalpellgriffs heben (Richtung ❶) und Klinge nach vorne abziehen in Richtung ❷.

- Lift the end of the scalpel blade above the lock of the scalpel handle (direction ❶) and remove the blade by pulling it forward in direction ❷.

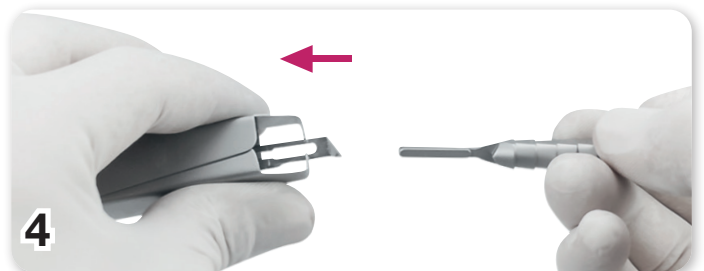

- Nach erfolgter Klingentnahme.
- After removal of scalpel blade.



- Entleeren des Klingentferners.
- Disposal of scalpel blade.

## Empfohlene Skalpellgriffe / Recommended Scalpel Handles



**46.007.00** Skalpellgriff **ZEPF**-Line, gerade, für Klagen # 10 -15, 14,5 cm

Scalpel Handle **ZEPF**-Line, straight, for blades # 10 -15, 14.5 cm



**46.007.01** Skalpellgriff, **ZEPF**-Line, abgewinkelt, für Klagen # 10 -15, 14,5 cm

Scalpel Handle, **ZEPF**-Line, angled, for blades # 10 -15, 14.5 cm



**46.005.65**