Pflegehinweise für Composite-Instrumente Handling Instructions for Composite Instruments

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Dear Customer!

We thank you very much for having chosen this high quality instrument from Helmut Zepf Medizintechnik GmbH.

To maximize the life of your Composite Instruments, please follow the instructions below:

- In all cleaning processes, it is very important that you always follow our instructions for reprocessing re-sterilizable manual instruments and their accessories in accordance with DIN EN ISO 17664 in the respective valid version. Refer to the document “0703091007_REPROCESSING_HZ_BA_e”, which can be downloaded from our website at www.zepf-dental.com.
- Any contact of the instruments with amalgam has to be avoided (e.g. when inlays are placed or due to residues on gloves).
- Remove residues of material on the instrument immediately, still in the dental chair, e.g. by wiping with a cloth soaked in alcohol, in order to avoid drying-up. Thus, you avoid subsequent manual effort which might possibly lead to a counterproductive scratching of the instrument surface.
- Prior to sterilization, dry the instruments carefully with a clean and soft cotton or microfiber cloth in order to avoid water spots.
- Do not use stainless steel brushes, brass brushes or abrasives for cleaning. To remove tenacious dirt, only use a firm toothbrush. Absolutely avoid any rough mechanical stress (abrasive detergents, e.g. detergent pads, cleaning cloths soiled by foreign particles, etc.).
- Avoid any aggressive cleaning agents (extremely acidic / basic, abrasive effect).
- If an ultrasonic cleaner is used, do not exceed the manufacturer's recommended ultrasonication period.
- After cleaning, rinse the instruments with clear or even better with demineralized water.
- If the instruments are cleaned in a disinfection bath, take care that the instruments' surfaces are not damaged by other instruments. A contact of the instrument parts during the cleaning process / during the sterilization has to be avoided (danger of contact corrosion).
- In thermal disinfection, we recommend to prepare these instruments in a washtray (see our sets **19.202.00**, **26.120.00TI** and **26.120.01TI**) in order to avoid damages on the surface (washtray without instruments: **85.181.00** and **85.182.00**).
- The instrument sets (washtray **19.202.00**, **26.120.00TI** and **26.120.01TI**) can smoothly be sealed in sterile film in which they can be stored up to 6 months (acc. to DIN 58953 Part 7).

Thank you for following these instructions which will maximize the lifespan of your instruments.