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onyx DESIGN CLEANING RECOMMENDATION FOR THE ZEPF onyx DESIGN COATING

The **onyx** coating is a hard material coating which positively influences the surface properties of the base material. Thanks to the combination of high hardness, low friction, improved corrosion resistance and low-glare surface, the coating is ideally suited for use in medical technology.

Nevertheless, the **onyx** coating also has its limits with regard to chemical resistance and coarse mechanical influences. For this reason, some recommendations for cleaning and care from our own experience and that of other users are presented to you in the following.

- Do not use any aggressive cleaning media (extremely acidic / alkaline, abrasive effect)
- Rinse with demineralized water after cleaning to prevent staining
- Rub dry with a clean cotton and microfiber cloth
- Always avoid coarse mechanical effects. Like abrasive cleaning agents, e.g. cleaning pads, soiled cleaning textiles (foreign particles), etc.

Additionally for medical technology:

- Avoid contact between parts during cleaning / sterilization
 - Risk of contact corrosion or material wear of soft instruments
 - Deterioration of the sterilization result
 - Store cleaned / sterilized parts dry

The **onyx** coating is applied to the substrate in a more gentle way than conventional methods. It is durable, hard and impact-resistant. It also displays outstanding anti-friction properties, which is a great advantage for the care of the instruments.

Drying on of proteins, etc. is not possible and any soiling can be wiped off.

Note: Even if a product's cutting edge does not appear black, the coating is still integrated in the surface.