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77R / # 77:

REF 17.677.16, 17.677.17

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[REF] 17.677.16, 17.677.17


Dear valued customer!

We thank you very much for having chosen the root elevator 17.677.16 and 17.677.17 for teeth no. 8, a high-quality instrument from Helmut Zepf Medizintechnik GmbH.

In order to enjoy this instrument for a long time, we kindly ask you to follow our application notes.

Explanation of symbols	
[MD]	Medical device
[REF]	Catalog number
[!]	Caution
CE	CE marking

MADE IN GERMANY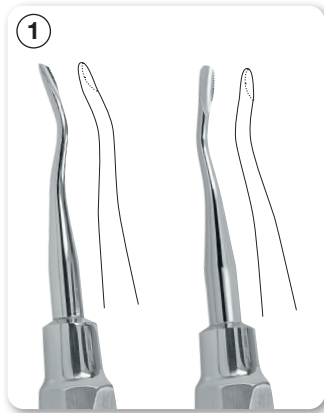


17.677.16


77R, ideal for wisdom teeth, mesial

17.677.17


77, ideal for wisdom teeth, distal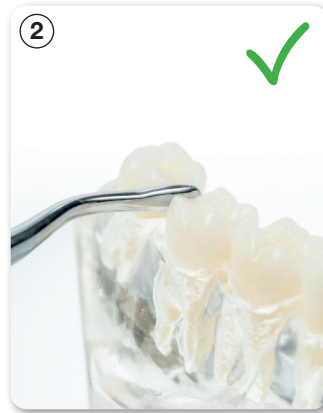

The bend of the instrument allows an excellent access on the approximal contact to root no. 8 in all quadrants. The fine serrated tip allows a gentle luxation.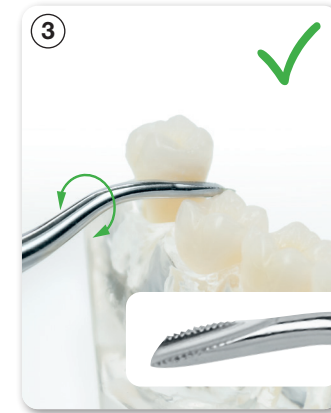


The fine serrated edges assure that the tooth will not slide from the instrument during luxation.

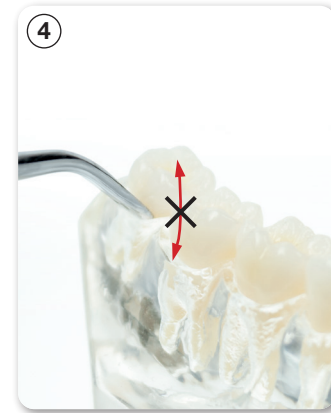Attention

It is not foreseen, to use the instrument as lever, as shown on the picture 4 or in any other way.

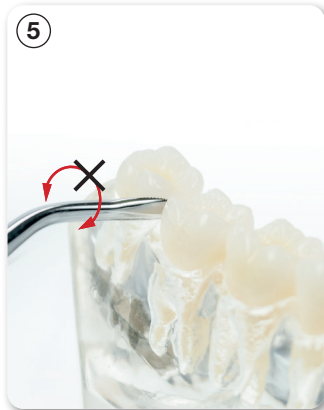
Attention

Please avoid any punctual overloading of the instrument on its absolute tip, as the working tip might break off.

We unfortunately cannot assume any liability for damages caused by improper use of the instruments.

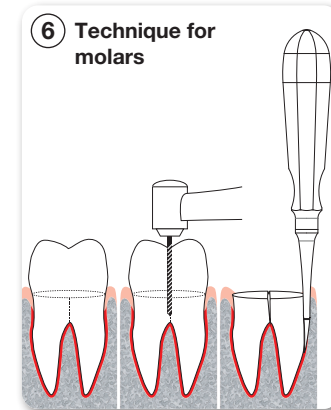6 Technique for molars

Cleaning and Reprocessing

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664, in the currently valid version. See document "0703091007_WIEDERAUFBEREITUNG_HZ_BA", downloadable from our website www.zepf-dental.com.

CE All medical devices indicated here bear the CE mark.

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