Kontrast-PA-Sonden-Einsätze

REF 24.451.00, 24.451.01, 24.451.02, 24.451.03, 24.451.06

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REF 24.451.00, 24.451.01, 24.451.02, 24.451.03, 24.451.06

Dear Sir / Madam,

Thank you for choosing the Contrast Periodontal Probes, which are high quality instruments from Helmut Zepf Medizintechnik GmbH.
To ensure that you can enjoy using these instruments for as long as possible, please follow our instructions for use.

1 GENERAL INFORMATION

The Contrast Periodontal Probes made of plastic are particularly suitable for use on implants.

Electrolyte reactions, such as oral galvanic currents triggered by salts dissolved in saliva, can be perceived as pain by the patient. This cannot happen with the Contrast Periodontal Probes made of plastic. In addition, scratching of the implants is also prevented.

2 ASSEMBLY / DISASSEMBLY

Always screw in the Contrast Periodontal Probe Inserts by hand (Figure A).

Not too hard, but gently to avoid breakages during cleaning and disinfection unit or during sterilizing. A typical example of this type of breakage can be seen in Figure B.

Please refer also to the section 4 STERILIZATION AND SERVICE LIFE.

 **Tool-free assembly.
Do not use tools!**

3 CLEANING

The Contrast Periodontal Probes are reprocessed manually or automatically in the cleaning and disinfection unit in accordance with the Robert Koch Institute guidelines. In the case of automatic reprocessing in the cleaning and disinfection unit, we also recommend unscrewing the Contrast PA Probes half a turn to promote longevity of the probe and to prevent the risk of it breaking off as a result of thermal expansion of the material.

In all cleaning processes, it is very important that you always follow our instructions for reprocessing re-sterilizable manual instruments and their accessories in accordance with DIN EN ISO 17664:2004. Refer to the document "0703091007_REPROCESSING_HZ_BA_e", which can be downloaded from our website at www.zepf-dental.com.

4 STERILIZATION AND SERVICE LIFE

The number of times that our Contrast Periodontal Probes can be used depends a great deal on the stresses / loads involved and on how they are reprocessed. The probes can be re-used until they are bent, bleached or no longer legible. If exposed to heavy stresses or loads, the Contrast Periodontal Probes made of plastic can be permanently bent out of shape. Attempting to bend the probe back into its original shape could promote or cause a break. For this reason, any Contrast Periodontal Probes that have become permanently deformed should be replaced.

Because our slender probe inserts are made of plastic and have a flexible working tip (Figure C), a permanent deformation can occur if the probe is sterilized under pressure or when already bent, or if the heat to which it is exposed exceeds 135 °C (275 °F)

 **Sterilize the Periodontal Probes lying down in a container without exerting any pressure on the working tip to avoid permanent deformation due to heat. In order to make them last longer, we also recommend unscrewing the Contrast Periodontal Probes half a turn before sterilizing them.**

4 DISPOSAL

Provided the local regulations are complied with, the instruments can be disposed of as normal household waste or in the recycling bin.