ZEPF Mikrochirurgischer Klingenhalter, Titan REF 46.013.00Z

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[www.zepf-dental.com/
anwenderinformationen](http://www.zepf-dental.com/anwenderinformationen)
CE 0483

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24.950.10

24.950.12

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**ZEPF Microsurgical Scalpel Handle,
Titanium REF 46.013.00Z
Dismantling, cleaning and maintenance**

www.zepf-dental.com/
en/user-information

CE 0483

Dear Customer!

We thank you very much for having chosen this high quality instrument from Helmut Zepf Medizintechnik GmbH. To get the best from your instrument, please follow the instructions below:

DISMANTLING

The microsurgical Blade Holder has to be dismantled easily **before** cleaning, as shown. Remove the back end completely by twisting the handle counterclockwise.

Pull-off the insert to the front.

CLEANING

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664.

See document "0703091007_REPROCESSING_HZ_BA_e", downloadable from our website www.zepf-dental.com.

Pre-Treatment: Use cleaning brush with nylon bristles when incrustations are heavy.

Cleaning: Manual or mechanical – ATTENTION: Cleaning in a thermal disinfectant: standing vertically, thread downwards.

Rinsing: Please use deionized water to avoid water spots.

Drying: Use compressed air or air-dry (manual preparation).

MAINTENANCE

Oiling with Medical White Oil, as shown

ASSEMBLY

Assembly is made in reverse order to the dismantling.

RECOMMENDED CARE PRODUCTS

Cleaning Brushes with nylon bristles

85.903.01 double-ended, sterilizable, packing unit: 3 pieces

85.903.03 for aspirator Ø 1.5 mm

85.903.04 for aspirator Ø 3 mm and Ø 5 mm

Medical white oil
in the 50 ml dosage bottle and in the oil pen, 12 ml
Physiologically harmless according to the German pharmacopoeia (DAB 10).

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