HELMUT ZEPF

MD Heber für axiale Luxation:
REF 17.006.01, 17.006.02,
17.006.03 und 17.006.04

CE 0483

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HELMUT ZEPF


MD Elevator for axial luxation:
REF 17.006.01, 17.006.02,
17.006.03 and 17.006.04

CE 0483

Dear valued customer!

We thank you very much for having chosen a high-quality instrument from Helmut Zepf Medizintechnik GmbH.

In order to enjoy this instrument for a long time, we kindly ask you to follow our application notes.

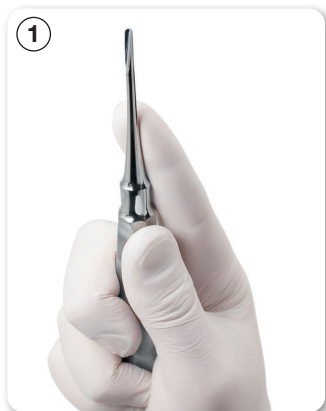For the best control, hold the elevator as illustrated.

Elevators for axial luxation are surgical instruments and should be used as such.

Do not use excessive force.

The instrument's tip is inserted into the sulcus towards the root in a most possible axial direction.

The tip is guided deeper along the root through gentle pressure and twisting movement in apical direction.

For optimal placement of the instrument, the tips are worked out more delicate.

Attention

Usage with leverage and non-planar pressure needs to be avoided.


This can cause breakage of the instrument tips.

Attention

We cannot be held responsible for damages caused by improper use.

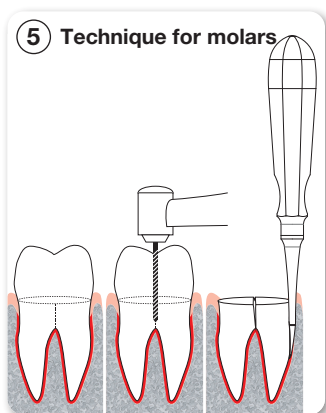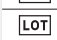
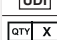
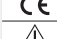
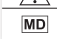
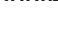
Note: These elevators for axial luxation have been developed as an alternative for classic elevators. The instrument must not be used as a lever.

Furthermore the gentle removal of the teeth should be reached by **axial luxation** and cutting the scharpey's fibres.

Cleaning and Reprocessing

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664, in the currently valid version.

The form is available for download on our website
"<https://www.zepf-dental.com/en/user-information/>".

Explanation of symbols	
	Observe the electronic instructions for use
	Manufacturer
	Date of manufacture
	Catalog number
	LOT no. (batch no.)
	Non-sterile
	Unique device identifier
	Quantity
	HIBC Barcode
	CE marking
	Caution
	Medical device

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Muito obrigado por ter escolhido um instrumento de alta qualidade da marca Helmut Zepf Medizintechnik GmbH.