HELMUT ZEPF

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REF 17.007.06LL - 17.007.07LL

REF 17.008.01LL - 17.008.02LL

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
REF 17.008.01LL - 17.008.02LL

Dear valued customer!

We thank you very much for having chosen a high-quality instrument from Helmut Zepf Medizintechnik GmbH.


In order to enjoy this instrument for a long time, we kindly ask you to follow our application notes.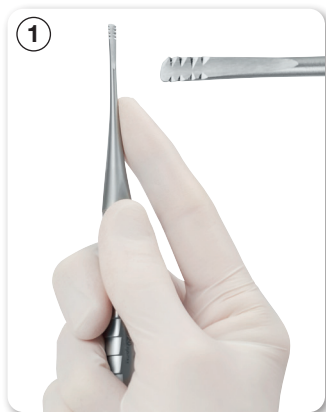


For the best control, hold the X-Tool as illustrated.

X-Tools are surgical instruments and should be used as such.

Do not use excessive force.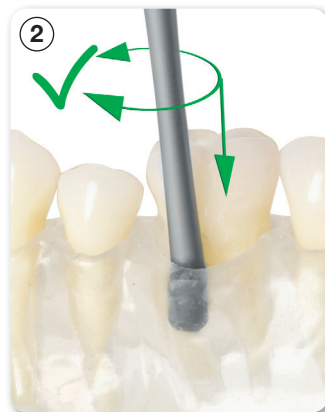

The instrument tip is inserted into the periodontal gap as axially as possible until the serration grips the root. The root is extracted by a barb effect with a straight pull. Provided the ligaments have been properly cut beforehand and the tooth has been loosened, the direction can be actively influenced by pulling (pulling effect).

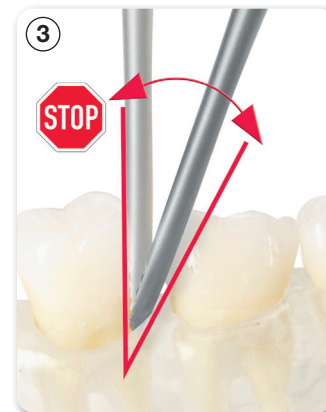For an optimal placement of the instrument, the tips are designed very delicate.

Attention

Usage with leverage and non-planar pressure needs to be avoided.


This can cause breakage of the instrument tips.

Attention

We cannot be held responsible for damages caused by improper use.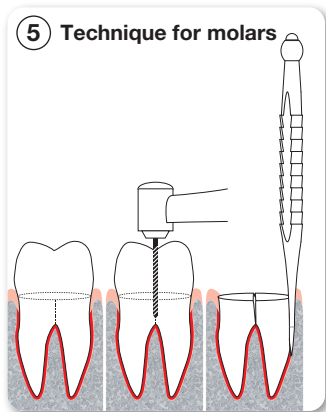

5 Technique for molars

Note: These X-Tools have been developed as an alternative for classic elevators. The instrument must not be used as a lever.

Rather, the gentle removal of the teeth should be reached by **axial luxation** and cutting the Sharpey's fibers.

Cleaning and Reprocessing

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664, in the currently valid version.

See document "0703091007_WIEDERAUFBEREITUNG_HZ_BA", downloadable from our website www.zepf-dental.com.

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Muito obrigado por ter escolhido um instrumento de alta qualidade da marca Helmut Zepf Medizintechnik GmbH.