ZEPF Bone Mill

REF 47.954.01

MONTAGE- & DEMONTAGEANLEITUNG

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ZEPF Bone Mill

REF 47.954.01

ASSEMBLING & DISASSEMBLING INSTRUCTIONS


Dear valued customer!

We thank you very much for having chosen the **ZEPF** Bone Mill 47.954.01, a high-quality instrument from Helmut Zepf Medizintechnik GmbH. In order to enjoy this instrument for a long time, we kindly ask you to follow our application notes.

APPLICATION

For crushing the bone material the handle has to be turned with appropriate effort. The result of the granulate can be influenced through various pressure on the pestle (direction of rotation see photo).

After using, the **HELMUT ZEPF** Bone Mill can be dismantled completely within 4 steps (without tools) and the sterilization and cleaning processes can be made.

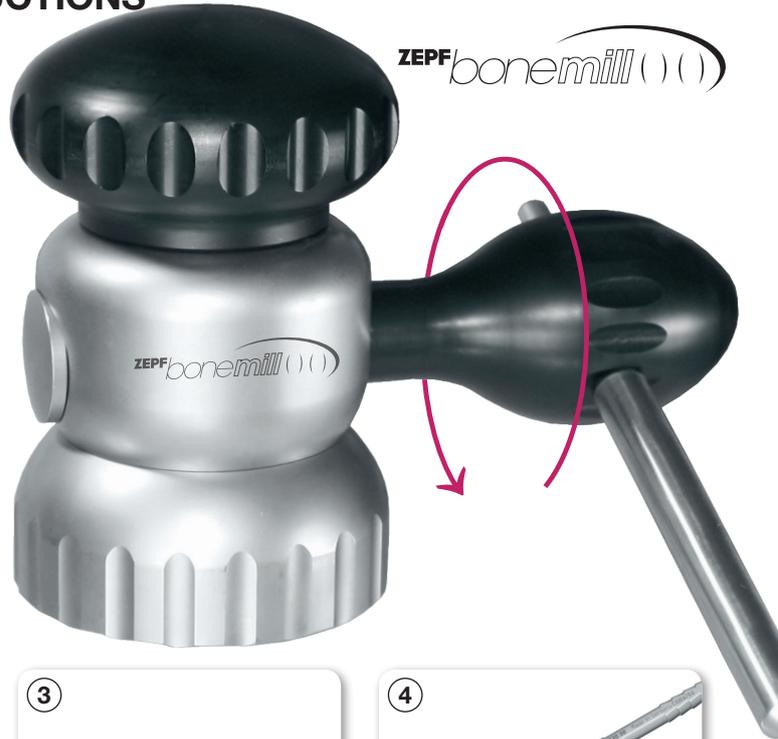Step 1: Take the lid / pestle and pull the handle element from the milling device.

Step 2: Remove the milling device, by pushing it through the filling corpus.

Step 3: Take the filling corpus from the collecting bank.

Step 4: Every single part can now be processed through the sterilizing and cleaning process.

CLEANING AND REPROCESSING

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664, in the currently valid version.

See document "0703091007_REPROCESSING_HZ_BA_e", downloadable from our website www.zepf-dental.com.