HELMUT ZEPF

Power Periotome

REF 26.690.01 - 26.690.03

REF 26.690.10 ZEPF FLEX-EX

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
REF 26.690.10 **ZEPF FLEX-EX**

Dear valued customer!

We thank you very much for having chosen a high-quality instrument from Helmut Zepf Medizintechnik GmbH.

In order to enjoy this instrument for a long time, we kindly ask you to follow our application notes.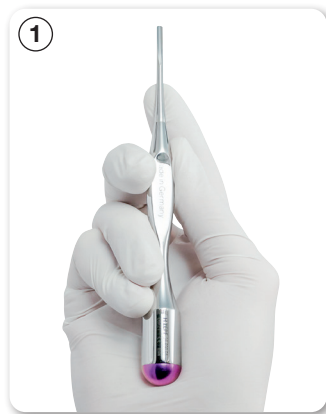

For the best control, hold the Power Periostome as illustrated.

Power Periostomes are surgical instruments and should be used as such.

Do not use excessive force.


Note: The **ZEPF** Power Periostomes allow a gentle loosening of ligaments in the sulcus. The instrument is not to be used as a lever.

The delicate working tips will enable you to build up a phenomenal pressure for the **luxation in a radial direction**.

The instrument's tip is inserted into the sulcus towards the root in a most possible axial direction.

The tip is guided deeper along the root through gentle pressure and twisting movement in apical direction.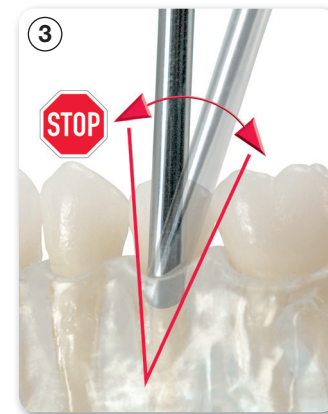

For optimal placement of the instrument, the tips are worked out more delicate.

Attention

Usage with leverage and non-planar pressure needs to be avoided.

This can cause breakage of the instrument tips.

Attention

We cannot be held responsible for damages caused by improper use.

CE 0483

Cleaning and Reprocessing

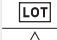
For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664, in the currently valid version.

See document "0703091007_WIEDERAUFBEREITUNG_HZ_BA", downloadable from our website www.zepf-dental.com.

0703091059_26690XX_POWER_PERIOTOME_HZ_BA | 13.04.2026 | 2 / 6

MADE  IN GERMANY

Explanation of symbols

	Observe the electronic instructions for use
	Manufacturer
	Date of manufacture
	Catalog number
	LOT no. (batch no.)
	Non-sterile
	Unique device identifier
	Quantity
	HIBC Barcode
	CE marking
	Caution
	Medical device

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