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BASIC CLEANING

ATTENTION: Before initial use and sterilization of the instruments, a basic cleaning has to be effected as follows:

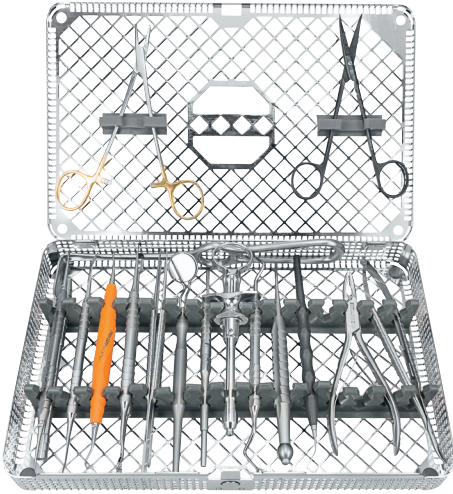
- Disassembly of dismountable instruments.
- Entire, bubble-free insertion in an appropriate detergent. We recommend a commercial dish-washing liquid, taking into consideration the manufacturer's information.
- Thorough mechanical cleaning of the entire instrument surface using a smooth plastic brush with nylon bristles (REF 85.903.01 packing unit 3 pieces).
- Rinse the instruments finally in desalinated (demineralized) water in order to avoid spots, residues and corrosion on the instruments.
- Thorough drying of the instruments before sterilization.
- Before initial cleaning, please read the attached instruction for the preparation of resterilizable instruments acc. to DIN EN ISO 17664.

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🇬🇧 ZEPF-Care Washbasket – the Space Wonder

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🇬🇧 PROBLEM: **Metal brushes** destroy the passive layer of the instrument surface!
SOLUTION: **Brushes with nylon bristles.**

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Cleaning Brush with nylon bristles

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