

LANGER M5 Universal Curettes in the versions:

Standard | Titanium | M5 | M5 with nanopal® coating | Delicate | Delicate with nanopal® coating
in ZEPF bionik handle



bionik




reddot design award
winner 2010


Langer Universal Currettes

in **ZEPF BIONIK** handle


Unlike Gracey Currettes, Universal Currettes have two working surfaces / cutting edges, so that concretions on the tooth neck as well as inflamed tissue in the pockets can be removed simultaneously in one step. The 3 instruments (Langer fig. 1/2, 3/4 and 5/6) allow an efficient prophylaxis treatment in all quadrants.

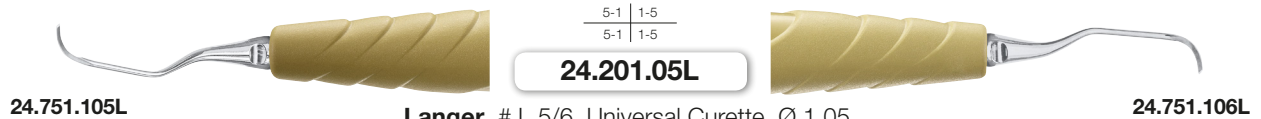

24.751.105L **Langer, # L 5/6, Universal Curette, Ø 1.05,** **24.751.106L**
for use on upper and lower front teeth, in **ZEPF BIONIK** handle yellow, exchangeable

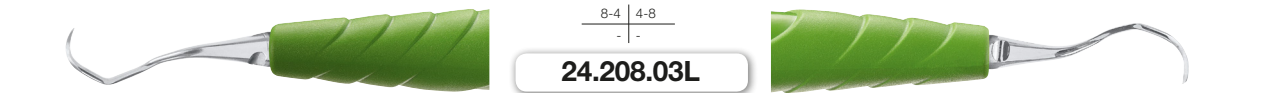

24.751.103L **Langer, # L 3/4, Universal Curette, Ø 1.05,** **24.751.104L**
for use on upper molars and premolars, in **ZEPF BIONIK** handle yellow-green, exchangeable

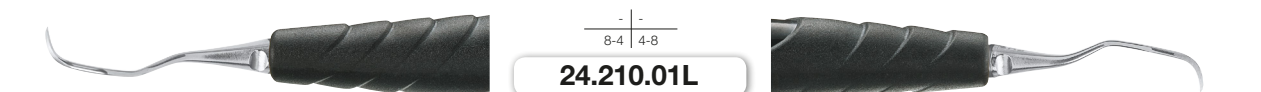


24.751.101L **Langer, # L 1/2, Universal Curette, Ø 1.05,** **24.751.102L**
for use on lower molars and premolars, in **ZEPF BIONIK** handle black, exchangeable




Langer Universal Titanium Currettes

in **ZEPF BIONIK** handle


The Langer currettes with titanium inserts are ideal for removing attached plaque layers on the implant necks. The titanium material does not damage the surface of the implant.

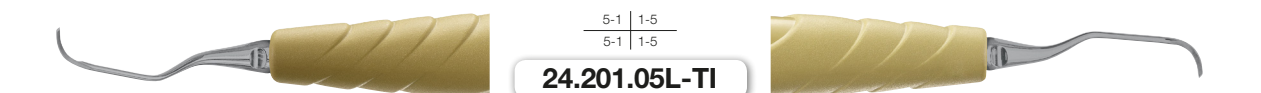

24.751.105L-TI **Langer, # L 5/6, Universal Curette with titanium inserts, Ø 1.05,** **24.751.106L-TI**
for use on upper and lower front teeth, in **ZEPF BIONIK** handle yellow, exchangeable

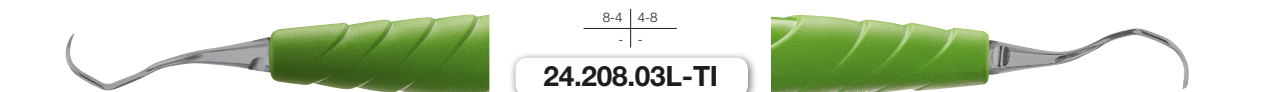

24.751.103L-TI **Langer, # L 3/4, Universal Curette with titanium inserts, Ø 1.05,** **24.751.104L-TI**
for use on upper molars and premolars, in **ZEPF BIONIK** handle yellow-green, exchangeable

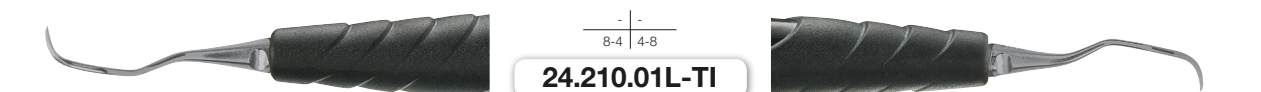




24.751.101L-TI **Langer, # L 1/2, Universal Curette with titanium inserts, Ø 1.05,** **24.751.102L-TI**
for use on lower molars and premolars, in **ZEPF BIONIK** handle black, exchangeable



Langer M5 Universal Currettes

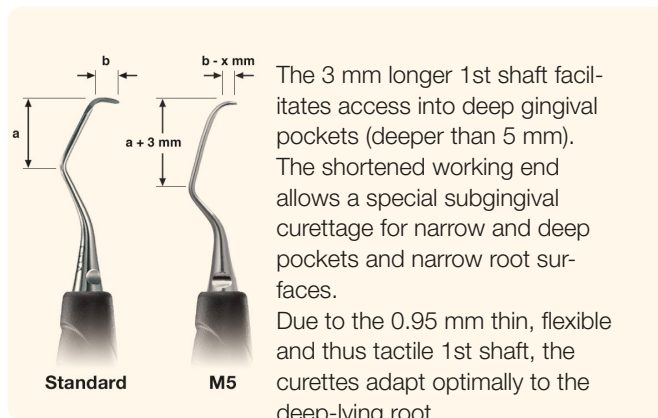

in **ZEPF BIONIK** handle

The new Zepf Langer M5 universal currettes are especially suitable for deep scaling – for removing deposits on tooth or root surfaces and simultaneously removing inflamed tissue. Due to the ground working end on both sides, the M5 Langer currettes are used especially for very deep pockets – in the closed periodontosis treatment.



	$\frac{5-1}{5-1} \mid \frac{1-5}{1-5}$	
24.751.105LM5	24.201.05LM5	24.751.106LM5
Langer, # L 5/6, M5 Curette, Ø 0.95, for use on upper and lower front teeth, in ZEPF BIONIK handle yellow, exchangeable		

	$\frac{8-4}{-} \mid \frac{4-8}{-}$	
24.751.103LM5	24.208.03LM5	24.751.104LM5
Langer, # L 3/4, M5 Curette, Ø 0.95, for use on upper molars and premolars, in ZEPF BIONIK handle yellow-green, exchangeable		

	$\frac{-}{8-4} \mid \frac{-}{4-8}$	
24.751.101LM5	24.210.01LM5	24.751.102LM5
Langer, # L 1/2, M5 Curette, Ø 0.95, for use on lower molars and premolars, in ZEPF BIONIK handle black, exchangeable		







Langer M5 Universal Currettes with nanopal® coating



in **ZEPF BIONIK** handle

The black-coated **ZEPF nanopal®** currettes have a durable, very hard surface which is very aggressive and sharp-edged due to its crystalline nanostructure. Due to the unique coating, the surface hardness is increased to an unprecedented **4500 Vickers**.

The cutting edge is thus highly tempered and does not need to be re-sharpened after each curettage. The black, scratch-resistant surface is easy to clean and prevents unpleasant light reflections.

	$\frac{5-1}{5-1} \mid \frac{1-5}{1-5}$	
24.751.105LM5OX	24.201.05LM5OX	24.751.106LM5OX
Langer, # L 5/6, M5 Curette with ZEPF nanopal® coating, Ø 0.95, for use on upper and lower front teeth, in ZEPF BIONIK handle yellow, exchangeable		

	$\frac{8-4}{-} \mid \frac{4-8}{-}$	
24.751.103LM5OX	24.208.03LM5OX	24.751.104LM5OX
Langer, # L 3/4, M5 Curette with ZEPF nanopal® coating, Ø 0.95, for use on upper molars and premolars, in ZEPF BIONIK handle yellow-green, exchangeable		

	$\frac{-}{8-4} \mid \frac{-}{4-8}$	
24.751.101LM5OX	24.210.01LM5OX	24.751.102LM5OX
Langer, # L 1/2 M5 Curette with ZEPF nanopal® coating, Ø 0.95, for use on lower molars and premolars, in ZEPF BIONIK handle black, exchangeable		

Langer M5 Universal Curettes, delicate version

in **ZEPF BIONIK** handle

Analogous to the standard Langer Universal Curettes, the delicate versions also have two working or cutting edges, so that concretions on the tooth neck and inflamed tissue in the pockets can be removed simultaneously in one step. Due to the 0.95 mm thin, flexible and thus tactile 1st shaft, they adapt optimally to the tooth necks.

	<table border="1" style="border-collapse: collapse; width: 60px; margin: auto;"> <tr><td style="padding: 2px;">5-1</td><td style="padding: 2px;"> </td><td style="padding: 2px;">1-5</td></tr> <tr><td style="padding: 2px;">5-1</td><td style="padding: 2px;"> </td><td style="padding: 2px;">1-5</td></tr> </table> <p>24.201.05LF</p>	5-1		1-5	5-1		1-5	
5-1		1-5						
5-1		1-5						
24.751.105LF	Langer , # L 5/6 Universal Curette, delicate version, Ø 0.95, for use on upper and lower front teeth, in ZEPF BIONIK handle yellow, exchangeable	24.751.106LF						

	<table border="1" style="border-collapse: collapse; width: 60px; margin: auto;"> <tr><td style="padding: 2px;">8-4</td><td style="padding: 2px;"> </td><td style="padding: 2px;">4-8</td></tr> <tr><td style="padding: 2px;">-</td><td style="padding: 2px;"> </td><td style="padding: 2px;">-</td></tr> </table> <p>24.208.03LF</p>	8-4		4-8	-		-	
8-4		4-8						
-		-						
24.751.103LF	Langer , # L 3/4, Universal Curette, delicate version, Ø 0.95, for use on upper molars and premolars, in ZEPF BIONIK handle yellow-green, exchangeable	24.751.104LF						

	<table border="1" style="border-collapse: collapse; width: 60px; margin: auto;"> <tr><td style="padding: 2px;">-</td><td style="padding: 2px;"> </td><td style="padding: 2px;">-</td></tr> <tr><td style="padding: 2px;">8-4</td><td style="padding: 2px;"> </td><td style="padding: 2px;">4-8</td></tr> </table> <p>24.210.01LF</p>	-		-	8-4		4-8	
-		-						
8-4		4-8						
24.751.101LF	Langer , # L 1/2, Universal Curette, delicate version, Ø 0.95, for use on lower molars and premolars, in ZEPF BIONIK handle black, exchangeable	24.751.102LF						







Langer Universal Curettes, delicate version with nanopal® coating



in **ZEPF BIONIK** handle

The black-coated **ZEPF nanopal®** curettes have a durable, very hard surface which is very aggressive and sharp-edged due to its crystalline nanostructure. Due to the unique coating, the surface hardness is increased to an unprecedented **4500 Vickers**.

The cutting edge is thus highly tempered and does not need to be re-sharpened after each curettage. The black, scratch-resistant surface is easy to clean and prevents unpleasant light reflections.

	<table border="1" style="border-collapse: collapse; width: 60px; margin: auto;"> <tr><td style="padding: 2px;">5-1</td><td style="padding: 2px;"> </td><td style="padding: 2px;">1-5</td></tr> <tr><td style="padding: 2px;">5-1</td><td style="padding: 2px;"> </td><td style="padding: 2px;">1-5</td></tr> </table> <p>24.201.05LFOX</p>	5-1		1-5	5-1		1-5	
5-1		1-5						
5-1		1-5						
24.751.105LFOX	Langer , # L 5/6, Universal Curette with ZEPF nanopal® coating, delicate version, Ø 0.95, for use on upper and lower front teeth, in ZEPF BIONIK handle yellow, exchangeable	24.751.106LFOX						

	<table border="1" style="border-collapse: collapse; width: 60px; margin: auto;"> <tr><td style="padding: 2px;">8-4</td><td style="padding: 2px;"> </td><td style="padding: 2px;">4-8</td></tr> <tr><td style="padding: 2px;">-</td><td style="padding: 2px;"> </td><td style="padding: 2px;">-</td></tr> </table> <p>24.208.03LFOX</p>	8-4		4-8	-		-	
8-4		4-8						
-		-						
24.751.103LFOX	Langer , # L 3/4, Universal Curette with ZEPF nanopal® coating, delicate version, Ø 0.95, for use on upper molars and premolars, in ZEPF BIONIK handle yellow-green, exchangeable	24.751.104LFOX						

	<table border="1" style="border-collapse: collapse; width: 60px; margin: auto;"> <tr><td style="padding: 2px;">-</td><td style="padding: 2px;"> </td><td style="padding: 2px;">-</td></tr> <tr><td style="padding: 2px;">8-4</td><td style="padding: 2px;"> </td><td style="padding: 2px;">4-8</td></tr> </table> <p>24.210.01LFOX</p>	-		-	8-4		4-8	
-		-						
8-4		4-8						
24.751.101LFOX	Langer , # L 1/2 Universal Curette with ZEPF nanopal® coating, delicate version, Ø 0.95, for use on lower molars and premolars, in ZEPF BIONIK handle black, exchangeable	24.751.102LFOX						

AESTHETIC IS THE RESULT